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EN

Double-sided and one-sided (L=coated on left side; R=coated on right side) diamond-coated oscillating metal strips.

Sterilizable and reusable.

To be used preferably in combination with the contra-angle:

- Intensiv Swingle: WG-69 LT, CH, WG-69 A.
- Intensiv Swingle SWISS EDITION: WG-69 LT S; WG-69 A S.

Intended use

Abrasion of dental material for reducing, contouring, finishing and/or polishing in Dentistry.

Application details

Initial opening of tight interproximal contact points. Unilateral or bilateral reduction, finishing and polishing of the interproximal surfaces during the IPR protocol in Orthodontics.

Instructions for use

- Application by dentist only.
- Apply Intensiv Ortho-Strips Opener for the opening of tight interproximal contact points.
- In case of tight contact points: open the interproximal space by placing a wooden wedge cervically.
- Subsequently apply Intensiv Ortho-Strips sequence according to the IPR protocol in Orthodontics.
- Ensure adequate cooling with water spray during the entire treatment.
- Insertion and removal of the strip in the interproximal space: insert and remove the strip in a cervical direction while the contra-angle is moving.
- Do not stop the strip in the interproximal space.
- It is advisable to fasten Intensiv Ortho-Strips with dental floss during treatment (loop around arch), for a better control of the instrument position.

Maintenance and Sterilization

- The strips are delivered non sterile. They must be sterilized prior to first use and disinfected, cleaned and sterilized after each use.
- Clean the strips and remove debris after each use, so as to maintain their abrasive properties.
- Sterilization must be carried out according to validated procedures.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch or visualized through the QR-code on the back page.

Risk Warnings

- Avoid bending and levering: use of excessive force (strong levering effect with the contra-angle) could lead to the breakage of the strip.
- Having passed over the tooth equator line with Intensiv Ortho-Strips, pay attention not to injure the soft tissues.
- Over 20'000 oscillations per minute could cause breakage of the strip.
- Avoid applying loads greater than 100 grams as this could cause damage to the strip and/or the tooth.

In the unlikely case of an incident caused by the instrument please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

Instructions for use and safety recommendations

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- Intensiv Swingle: WG-69 LT; CH: WG-69 A
- Intensiv Swingle SWISS EDITION: WG-69 LT S; WG-69 A S.

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